

INSTRUCTIONS FOR COMPREHENSIVE STOOL ANALYSIS TEST

Please read instructions completely before you begin.

OBJECTIVE: The collection of two soft stool specimens passed 2-3 days apart or when the patient is most symptomatic.

RESTRICTIONS AND PRECAUTIONS:

1. The lighter the color of the stool, the better. If possible, adjust your diet during the same period by eating more fiber to produce a soft specimen.
2. Allow at least two weeks after termination of medication before collecting stool specimen, Medications such as antibiotics, anti-parasitic, anti-malarials, mineral oil, antacids, non-absorbable anti-diarrheal preparations, or bismuth, e.g. Pepto Bismol, can affect the accuracy of the results as well as probiotics. Do not use castor oil or liquid petrolatum. Do not collect the sample after undergoing radiological examination involving barium sulfate. **Do not discontinue medications without your physician's approval.**
3. Handle with care, each specimen represents a potential source of infection with parasites, bacteria or viruses.
4. If any liquid preservative from the vials or stool specimen gets into eyes or on skin, immediately wash area with water for at least 15 minutes. Avoid breathing vapors. If any discomfort or irritation develops, contact a physician. **The preservative is toxic if swallowed & irritating to eyes and skin. Antidote:** rinse exposed area with water, then wash with soap and water. If swallowed, do not induce vomiting; give plenty of water to drink and seek medical attention. **Active ingredients:** 2-Bromo-2-Nitro-1,3-Propanediol and Zinc Sulfate and includes no carcinogenic chemicals. The preservative is environmentally friendly and does not include mercury, formalin or alcohols.
5. In case of constipation, non-absorbable high fiber, herbal laxatives may be used to produce softer stool specimens as long as they do not contain any anti-parasitic or anti-infectious ingredients. We recommend Miralax as a laxative.

DIRECTIONS:

1. Carefully open one vial containing preservative. Using the spoon attached to the cap of the vial, collect small amounts of stool from near the top of your specimen and especially from areas that are slightly watery or bloody, if present, and place into vial. If diarrhea, collect the latter part of stool specimen. The stool sample kit is biodegradable. Flush with stool when done. **IMPORTANT: FILL VIAL WITH JUST ENOUGH SPECIMEN TO REACH THE FILL LINE. DO NOT FILL OVER LINE. OVERFILLING CAN AFFECT THE ACCURACY OF RESULTS.**
2. Replace and tighten the spoon/cap and shake vial until contents are thoroughly mixed. Make certain that vial is tightly capped to avoid leakage. Vial does not need to be refrigerated. Fill out label on vial. Place into Ziploc bag in which the preservative vials were originally shipped to you. You do not need to fill out the ID number on the vial that is a number we will give it.
3. Repeat the procedure two or three days later and obtain another stool specimen, then add the second vial into the same Ziploc bag. If your symptoms are cyclical (sometimes they are worse and then better) it is more important to take your specimens when you are feeling worse than following the general 2-3 days apart guideline. If you have a history of negative results, but still have symptoms you can take a total of 4 smaller specimens (2 in each vial), every other day. This way we will have 4 different days of data spaced out over a week.
4. Fill out requisition form completely. **Samples not accompanied by the completed requisition form will NOT be processed.** If you want to use **Mastercard, Visa, or Discover (we do not accept American Express)**, include the credit card information on the requisition form. You may use whatever courier you like to send the specimens back to us (USPS, UPS, FEDEX etc.). Store in a cool place before transport
5. **Send to PCI at 11445 E. Via Linda #2-419, Scottsdale, AZ 85259**

You can receive your results directly from your patient profile; if you didn't register as a patient and only signed out as a guest, go to forget password, and then sign in, you will see your test under the tab reports. Stool results will be available within 3-5 business days of receipt of the sample, not including weekends or holidays.