

INSTRUCTIONS FOR PARASITE ID TEST

The vials contain 4% buffered formalin for collection, preservation and transport of biological specimens to PCI for diagnosis.

1. Include only clean specimens (worms, worm parts or any other biological material only from your body) free from any debris, fecal matter, blood or any other secretions or contaminants.
2. Seal tightly to prevent leakage.
3. Completely fill out the required information on the Requisition Form and on the label on the vial. Samples not accompanied by the completed Requisition Form will NOT be processed. This lab is NOT a Medicare provider and cannot accept Medicare patients.
4. Package properly and ship using US postal service or any other mail carrier of your choice. Diagnostic results will be faxed directly to the practitioner indicated on your Requisition Form.