

Parasitology Center, Inc.

11445 E. Via Linda, # 2-419

Scottsdale, AZ 85259 Ph: (480) 767-2522 Fax: (480) 767-5855

[www.parasitetesting.com](http://www.parasitetesting.com)

---

## INSTRUCTIONS FOR PINWORM TEST

1. Holding the cap, remove paddle from the container, do not touch the paddle
  2. Separate the buttocks and press the sticky surface of the paddle against the anal area. The specimen should be obtained early in the morning before arising or bowel movement.
  3. Return the paddle to container and secure cap tightly.
  4. Wash hands thoroughly with soap after you have finished.
  5. Print patient information on the label.
  6. Results will be made available to your health care practitioner within 2-5 working days.
-