

PARASITOLOGY CENTER, INC. (PCI)

11445 E. Via Linda, #2-419

Scottsdale, AZ 85259

Ph. (480)767-2522

Fax (480)767-5855

www.parasitetesting.com

INSTRUCTIONS FOR URINE TEST

Please read instructions completely before you begin.

1. Unscrew cap to container provided.
2. Do not remove preservative from container, **the preservative is toxic if swallowed & irritating to eyes and skin. Antidote:** rinse exposed area with water, then wash with soap and water. If swallowed, do not induce vomiting; give plenty of water to drink and seek medical attention.
3. To reduce contamination, the specimen submitted for analysis should be a clean catch “mid-stream” sample.
4. Provide 50 ML of urine in addition to the 40 ML of preservative into the container and re-screw the lid onto the container.
5. Fill out your name, last 4 of social security number, and date on label provided.
6. Mail back to PCI for testing.
7. Results will be ready within 2-5 business days of receipt of sample, not including weekends and holidays.

If you have any questions please use the “contact us” link on our website www.parasitetesting.com

You can receive your results directly from your patient profile; if you didn't register as a patient and only signed out as a guest, go to forget password, and then sign in, you will see your test under the tab reports.